

AMSure® Sal-Sense™

Single Use Saline Topical Solution 0.9% Sodium Chloride

Introducing Sal-Sense, a single-use, cost-effective solution to clean wounds, cuts, and minor burns

- Single patient use avoids cross contamination.
- Save time from standard practice of drawing solution.
- Easy transport in your pocket to patient.
- Twist design to open effortlessly.
- Easy to irrigate area in any position.
- Perfect solution for Long Term Care, Hospitals, Primary Care, Home Health, EMS, and more...



Item Code	Description	Qty/Unit
AS0159	15 mL Single Use Saline Topical Solution	144/Case—3 Boxes (48 per Box)
AS0359	35 mL Single Use Saline Topical Solution	120/Case—6 Boxes (20 per Box)
AS0559	50 mL Single Use Saline Topical Solution	120/Case—6 Boxes (20 per Box)

Certificate of Compliance will be produced for each batch manufactured.

AMSINO® Sal-Sense™

Single Use Saline Topical Solution 0.9% Sodium Chloride



Simplify your irrigation process with one step

INDICATIONS FOR USE: For use in moistening and lubricating absorbent wound dressings for cuts, bruises and minor burns prior to removal from the wound area.

- Do not use unless the product is clear.
- If there is any evidence of leakage prior to breaking the seal, discard. Discard any unused portion.
- Does not contain antimicrobial or other ingredients.
- For external use only. Not for injection.
- Do not use for wounds that are exposed to cerebrospinal tissue.
- Do not place the device on sterile field.
- Maximum use 59 mL/day for infants and 169 mL/day for children.

DIRECTIONS FOR USE: Using aseptic technique.



1. Separate a single vial from group, tearing from top down (not bottom up).

2. Use undiluted.

3. Twist off the top and discard the top cap.

4. Apply topically to the affected area with a firm pressure on the vial providing a steady stream of the solution. To obtain drop by drop dispensing, invert the vial and squeeze gently.

5. For single patient use. Discard any remaining contents.

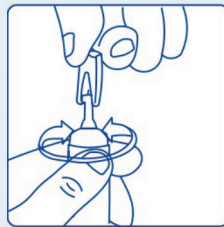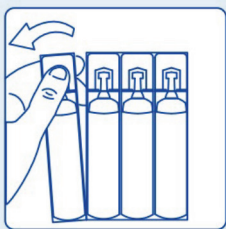

AMSINO
Advancing Healthcare Worldwide®

Amsino Medical Group Co. Ltd.

708 Corporate Center Drive, Pomona, CA 91768 USA

+1 800.632.6746 • fax: +1 909.626.3888 • info@amsino.com

To view our complete product line, visit us at www.amsino.com

Product(s) described may not be licensed or available for sale in all countries. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Items listed with a CE mark are legally available within the European Union (EU) and other regions where CE marking may be required. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. AMSure and Sal-Sense are registered trademarks of Amsino Medical Group Co. Ltd. © Copyright 2023, Amsino Medical Group Co. Ltd. All rights reserved.