AMSINO

**ENG: Blood transfusion set with needle
Filter 200 µm
Needle 18G**

Intended purpose

Blood transfusion set with needle is used to transfuse blood to patient.

Indication

The Blood transfusion set with needle is intended for the intravenous infusion of whole blood, packed red blood cells, platelets, plasma, cryoprecipitate, and other blood components.

Contraindication

- Presence of infection at the intended site of needle insertion.
- Coagulopathy.
- Sepsis.

- Thrombocytopenia.

Intended users

- Physicians, physician's assistants at all levels.
- Nurses at all levels.

Intended patient population

The device is intended for adults and elderly people in operating rooms, emergency clinics and inpatient wards. Vulnerable patients including pregnant women and lactating women should use this product with caution under the guidance of medical professionals.

Direction For Use

1. Peel package open and remove set.
2. Close clamp.
3. Remove protector from spike end.
4. Insert the spike into a blood bag.
5. Squeeze and release drip chamber until half filled.
6. Remove protector from lower end.
7. Open clamp partially to prime set. Prime set and purge air from tubing. Close roller clamp.
8. Perform venipuncture.
9. Regulate flow with clamp.
10. Check maintenance of proper flow rate at regular intervals.

Warning:

- Do not use the product if the package is damaged or unintentionally opened before use.
- Destroy after single use. Dispose of this product in a safe manner according to local guidelines for disposal of contaminated waste.
- Do not allow air to be trapped in set.
- Do not use it if tip protectors are not in place.
- Not for use in infusion pumps.
- For gravity use only.
- Use it under the guidance of doctors to prevent hemolysis.
- Federal (USA) law restricts this device to sale by or on the order of a physician.

Storage condition: Avoid freezing and excessive heat.

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Amsino International, Inc.