

Instruction for use

REF iRM500

Intended Purpose

The iReceptal Mini Digital Surgical Suction System is intended to be used in the operating room, pathology, surgical centers, and doctor's offices to collect and dispose of surgical fluid waste as well as collect smoke generated from electrocautery or laser devices.

Contraindications For Use

The iReceptal Mini Digital Surgical Suction System is contraindicated against:

- Connection directly to chest tubes.
- Connection to closed wound drainage systems.

For Use With

The Disposable Manifold (manifold) is used with the iReceptal Mini Rover. The Disposable Manifold is non-sterile, single patient use, and disposable.

The Disposable manifold provides a fluid path from the suction lines to the rover canister for the collection of fluid waste. Standard off-the-shelf commercially available suction tubing is connected to the manifold.

The Disposable manifold has a built-in collection basket that can be used as a tissue specimen collection. This collection basket can be accessed by opening the disposable manifold's lid.

Safety Directives

WARNING: Do not use a disposable manifold that has been dropped or mishandled.



Failure to comply may cause leakage and result in patient and/or healthcare staff injury.

WARNING: The disposable manifold is for single patient use only. Do not reuse, reprocess, service, modify, or repackage a single use device.



- The single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
- Design features may make cleaning difficult.
- Reuse may create a contamination risk and compromise structural integrity, resulting in operational failure during use.

Failure to comply may lead to infection or cross-infection and result in patient and/or healthcare staff injury.



WARNING: The Exposure Control Plan is designed to eliminate or minimize employee exposure through use of personal protective equipment (PPE), appropriate vaccinations (e.g. hepatitis B), and other control measures.

- Always wear PPE when operating or handling this equipment.
- Always leave suction tubing attached to the disposable manifold and close unused ports during use and disposal.

Failure to comply may cause infection and result in healthcare staff injury.

Instructions

To Install the Disposable Manifold

Insert the disposable manifold into the manifold receptacle and push the disposable manifold downward until you feel it click into place. Use 5/16 in. [7.9 mm] inner diameter suction tubing or 1/4 in. [6.4 mm] inner diameter suction tubing with a 5/16 in. [7.9 mm] adapter, as required. Close all the unused disposable manifold ports using their corresponding caps.

To Remove the Disposable Manifold

Press the disposable manifold ejector to release the disposable manifold from the receptacle.

Lift the disposable manifold and attached suction tubing out of the receptacle keeping the disposable manifold in a horizontal orientation.

To Use the Disposable Manifold with Specimen Collection

Remove the built-in collection basket from the disposable manifold by opening the manifold's lid, if a specimen is collected and transfer of the specimen is required.

Shelf Life: 3-years



Amsino International, Inc.
708 Corporate Center Drive
Pomona, CA 91768, USA
www.amsino.com



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



Made in China

L11491 Rev.C ENG