

RECEPTAL™ Canister and Liner Set-up

Single Canister and Liner Set-up



1 Place canister into support.



2 Insert same size liner with VAC-GARD® and secure liner lid to canister.



3 Connect lid VAC-GARD® blue tubing and vacuum tubing to canister "T".



4 Connect patient tubing to patient port. Cap all ports not in use.

Tandem Canister and Liner Set-up



1 Select canister size and support. Place canisters into support.



2 Connect canister "T"s with vacuum tubing.



3 Insert same size liners (1 with and 1 without VAC-GARD®), secure liner lids to canisters.



4 Connect lid VAC-GARD® blue tubing and vacuum tubing to canister "T".



5 Connect lid tandem tubing (clear) to patient port of the lid with blue tubing.




6 Connect patient tubing to patient port. Cap all ports not in use.

Tandem Wall Mount Set-up



AMSINO

Advancing Healthcare Worldwide®

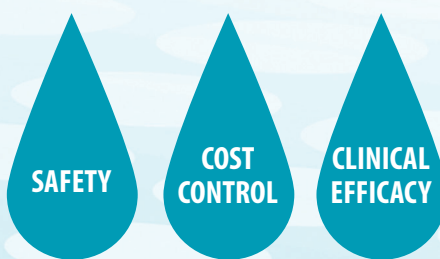

Amsino International, Inc.

708 Corporate Center Drive, Pomona, CA 91768 • 800.632.6746 • fax: 909.626.3888 • info@amsino.com

Product(s) described may not be licensed or available for sale in all countries. Please see the instructions for US/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. RECEPTAL is a registered trademark of Amsino Medical Group Holding Co. © Copyright 2016, Amsino International, Inc. All rights reserved.

WFM-GLO-2016-06-15A